

DESIGNED FOR SUPPORT AND PERFORMANCE

Back-up support and shape retention to
help you through your challenging cases.




VISTA BRITE TIP™
GUIDING CATHETER



DESIGNED FOR SUPPORT AND PERFORMANCE

Support

BACK-UP SUPPORT AND TORQUE CONTROL

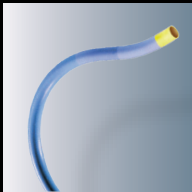

- Hybrid braid construction delivers support, torque control and lumen integrity.
- Multi-segment design balances strength and flexibility promoting coaxial alignment, kink resistance and support.


SHAPE RETENTION



- Designed to maintain shape in lengthy cases and challenging procedures.


Performance

EASY DELIVERY OF DEVICES

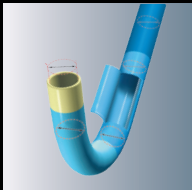

- PTFE lining provides ease of device delivery.
- Consistent diameter from hub to tip.

ENHANCED VISIBILITY

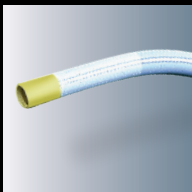

- Soft, radiopaque tip facilitates atraumatic engagement.

FEATURES

Hybrid braid

Multi-segment design / radiopaque tip

PTFE liner


Consistent inner diameter

Guiding Catheters are available in a wide range of shapes and configurations.



HYBRID BRAID CONSTRUCTION

- Proprietary hybrid braiding
- Allows for a thin wall without sacrificing support
- Delivers excellent
 - Support
 - Torque control
 - Shape retention

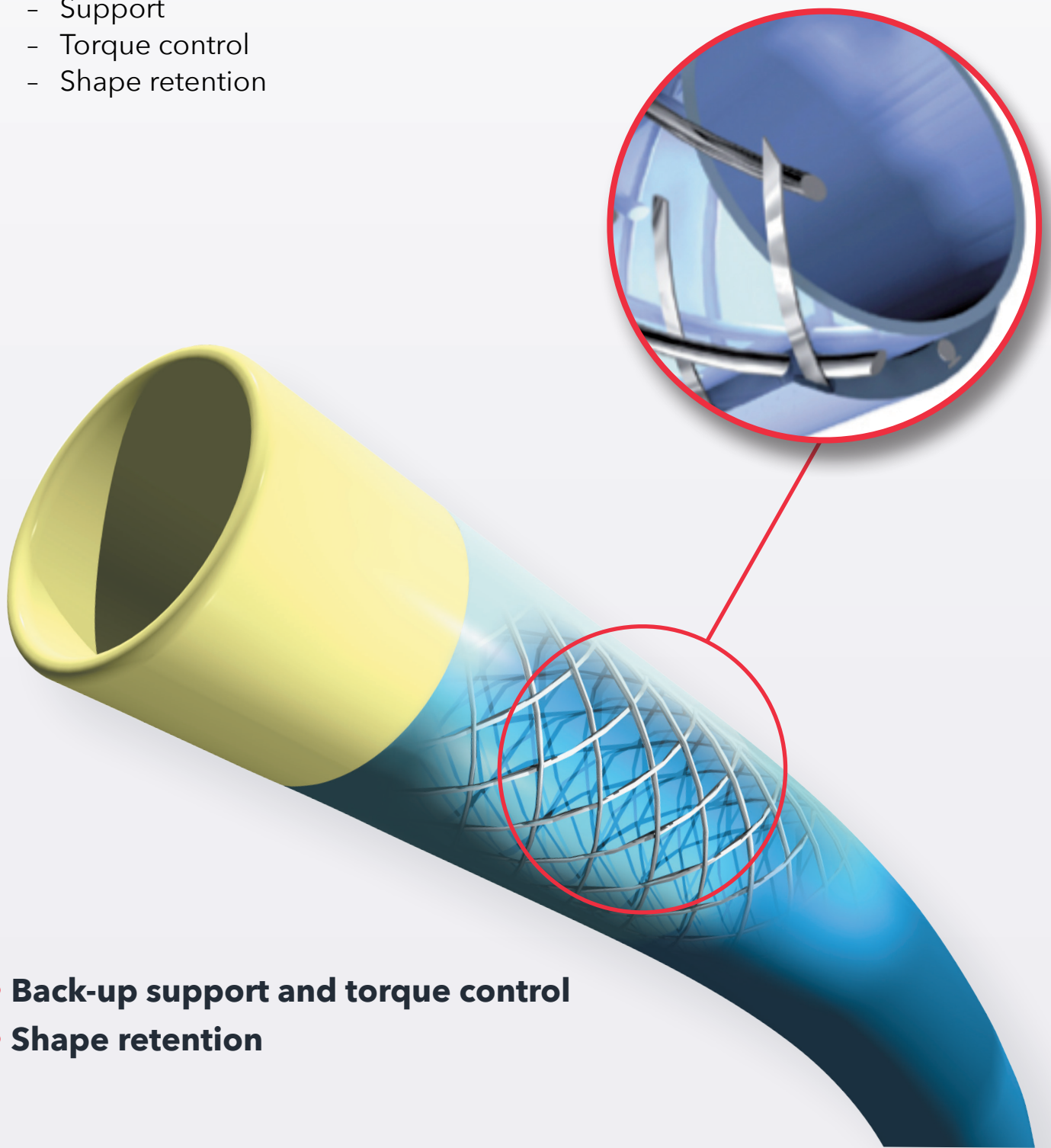

- **Back-up support and torque control**
- **Shape retention**

MULTI-SEGMENT DESIGN

A variation of design and materials to balance strength and flexibility promoting:

- Coaxial alignment
- Kink resistance
- Support

Kink Resistant Segment

The segment absorbs the kink forces between the firm shaft and the soft distal segments.

The Bright Tip

Soft Brite Tip helps ensure atraumatic cannulation.

Coaxial Segment

This soft braided segment optimizes tip flexibility and provides gentle cannulation.


Torque/Support Segment

Torque support segment delivers responsiveness from hub to tip.

- **Back-up support and torque control**
- **Shape retention**
- **Excellent visibility**

VISTA BRITE TIP™ Guiding Catheter

PTFE Lining and true lumen design provide consistently lubricious lumen from hub to tip for smooth delivery of devices.

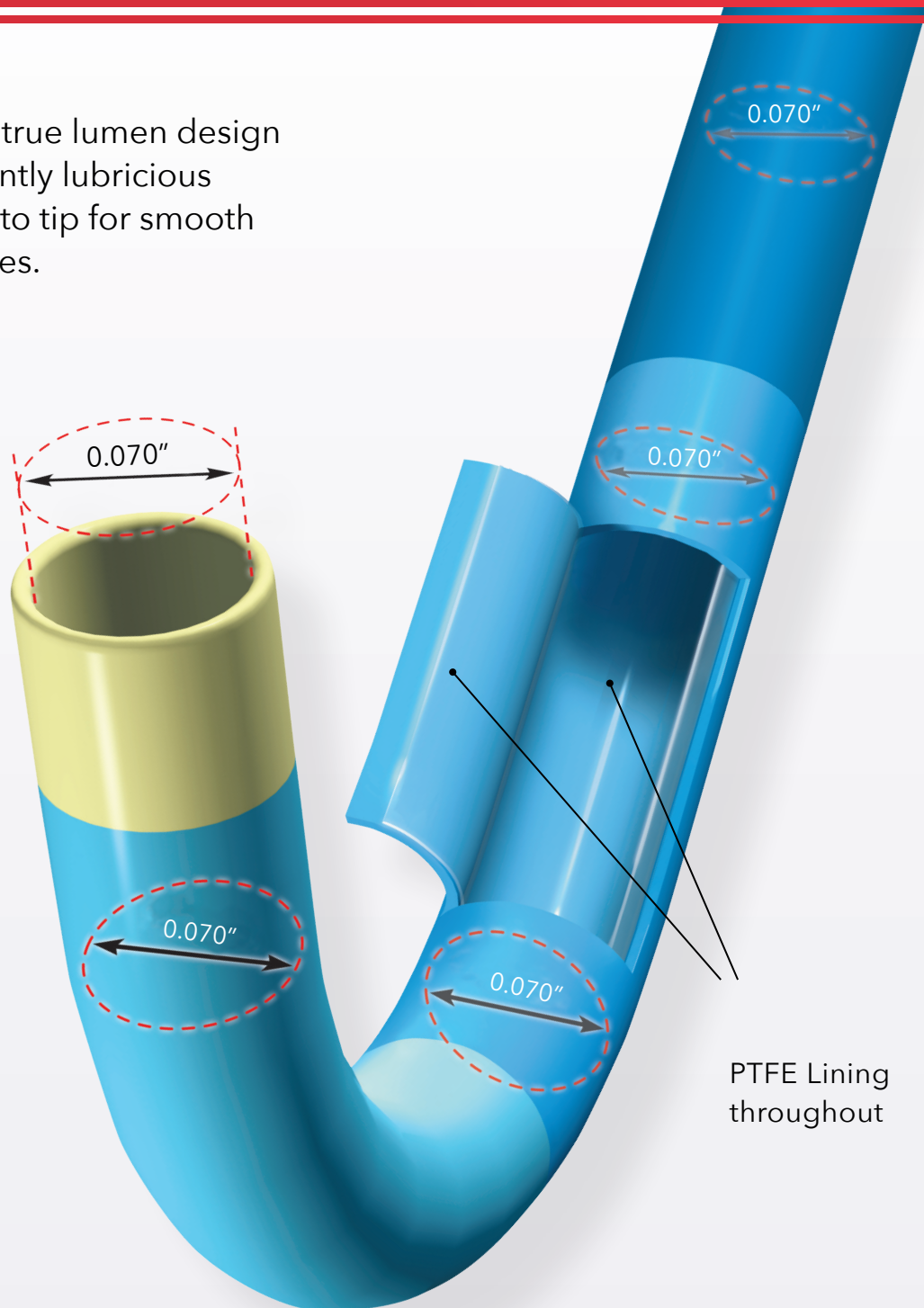

Consistent diameter from hub to tip



- **Easy delivery of devices**

LONG VISTA BRITE TIP™

Guiding Catheter

Long VISTA BRITE TIP™ Guiding Catheter provides atraumatic, deep intubation for conformity with the anatomy of vessels.


- 16 mm tip
- Deep atraumatic cannulation

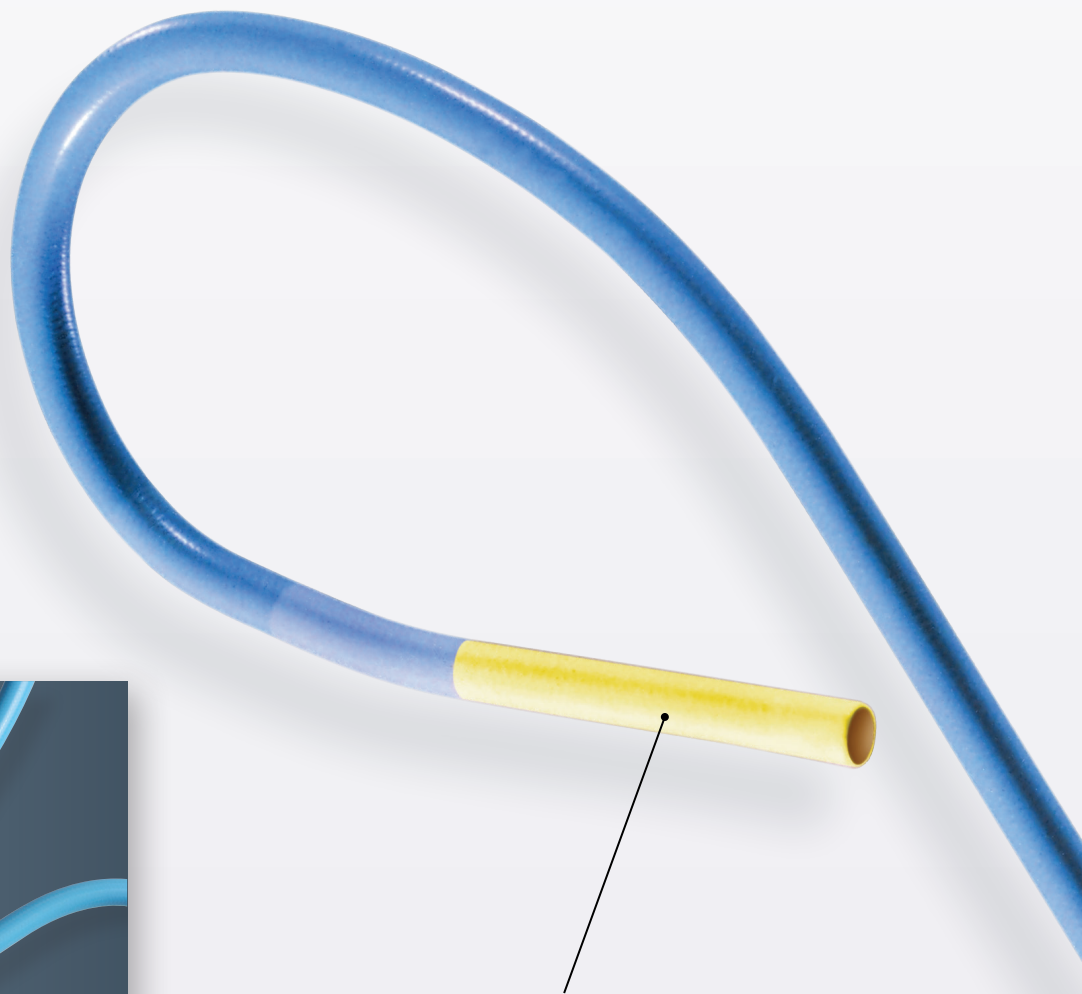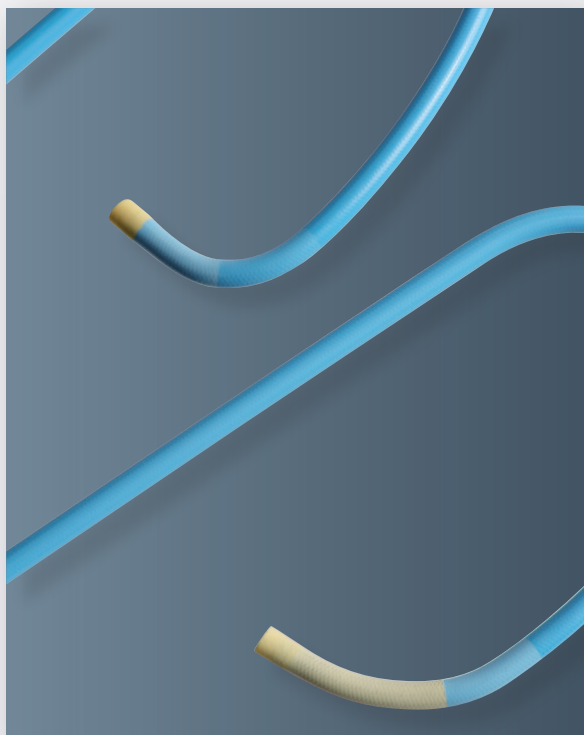

16 mm ultra soft tip
facilitates active
support

GUIDES FOR BOTH THE LEFT AND RIGHT CORONARY ARTERIES



LCA

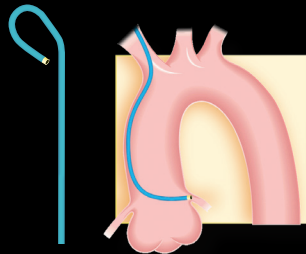

XB

First choice guiding catheter, adapted to most anatomies which allows very good coronary support and with minimal trauma.



- More backup support than JL
- Allows good coaxial alignment to intubate the circumflex and left anterior descending (LAD) arteries
- Adapted for normal ostium take-off
- 1/2 size down from JL standard (XB 3.5 = JL4)
- Suited for transradial procedures, including XB-2.5

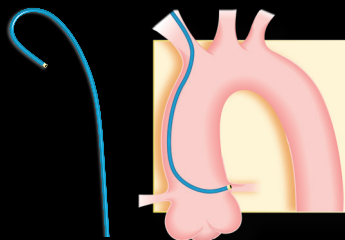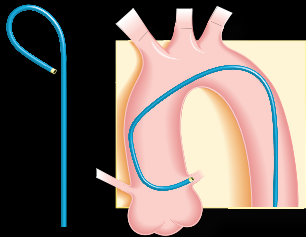

XBC

An extra backup shape for the left coronary and adapted to a short or absent left main and to a very angular CX origin.


- More backup support than JL
- Adapted for normal, superior or inferior aortic take-offs
- 1/2 size down from JL standard (XBC 3.5 = JL4)
- Suited for transradial procedures

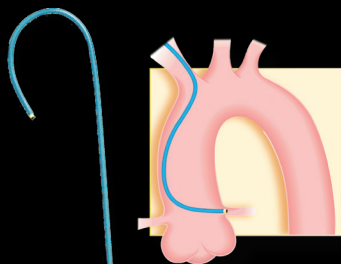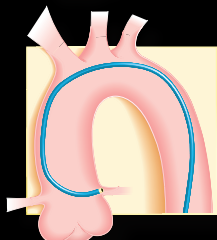


XBLAD

Extra-support for the intubation of the left coronary and more specifically designed for the LAD.

- More support than a JL4 and enables extra support for LAD
- Adapted for superior aortic take-offs
- 1/2 size down from JL standard (XBLAD 3.5 = JL4)
- Suited for transradial procedures

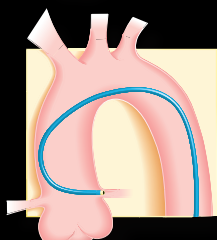


RCA

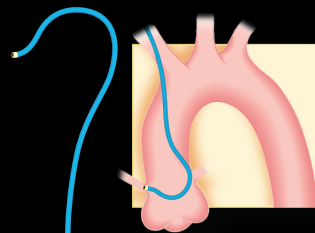

XBRCA

The Extra Back up shape designed for the Right Coronary.


- More support than an Amplatz or JR shapes
- Adapted for normal or superior aortic take-offs

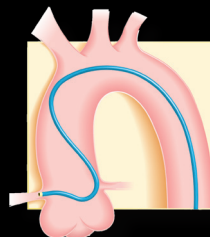


**Extra-Back up
shapes available**

EXTRA BACK-UP IN TRANSRADIAL INTERVENTIONS

Judkins Fajadet Right (JFR)

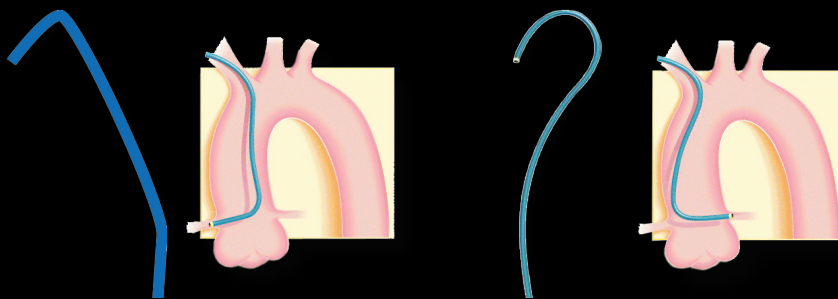

A modification of the Judkins catheters especially developed for giving extra back up during transradial procedures.

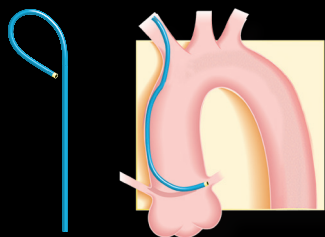

XB 2.5

In general, engagement of the left coronary artery requires smaller catheters (i.e. a JL 3.5 compared with a JL 4) compared with those used for femoral or left radial approaches. XB-2.5 is a smaller curve of the best selling XB family. Suitable for aortas in which JL-3.0 would be needed radially.

LONG Catheters


Cordis offers a number of guiding catheters longer than the standard 100 cm for:

- very tall patients
- tortuous anatomy
- angioplasty of the renal artery using right or left radial approach

RADIAL SOLUTIONS: SPECIAL RADIAL SHAPES

Radial Bi-Lateral / Multipurpose

Cordis offer a series of catheter choices to cannulate both the LCA and RCA.

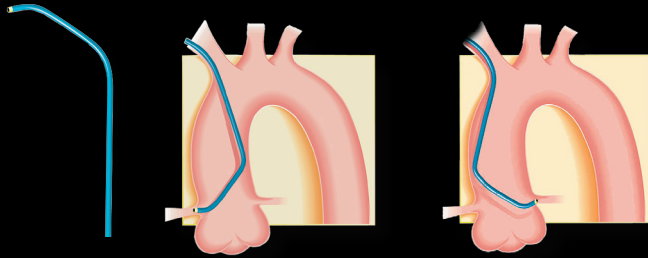


Barbeau

Coronary catheterisation via right radial access of the left and right coronary arteries.

- Primary curve of 135 degrees added to an MPA catheter (Multipurpose A)
- Can be used for the right and left ostium cannulation
- Curvature very suitable for the right coronary artery
- If the left coronary artery is difficult to cannulate use a XB catheter


ANGIOPLASTY OF RENAL ARTERY USING LEFT OR RIGHT RADIAL ACCESS

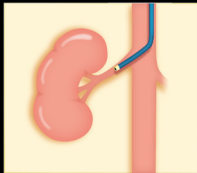

MPA 125 cm Curve / JR4 125 cm

- Allows cannulation of ostiums of descending arteries
- Alternative for patients with no femoral access

Other shapes

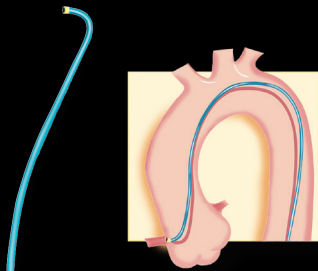


3DRC (Williams Technique)

A popular choice of guiding catheter, suitable for most anatomies of the right coronary.

Three-dimensional curve allows for limited torque needed for its insertion in the right coronary artery. Suitable for:

- ostial lesions
- right and left internal mammary arteries

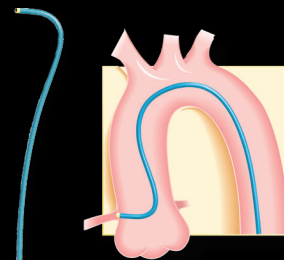


Hockey Stick - HS Curve

Guiding catheter which are able to catheterise the right coronary artery and right saphenous vein bypass grafts.

The HS offers more support than a Judkins right (JR) and can be a first choice catheter.


Adapted to radial approach.

Other Options for PCI in Coronary Bypass

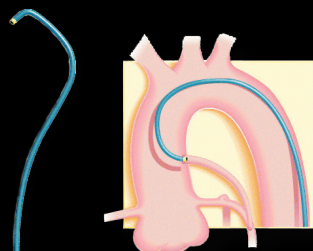


LCB

Left Coronary Artery via Femoral.

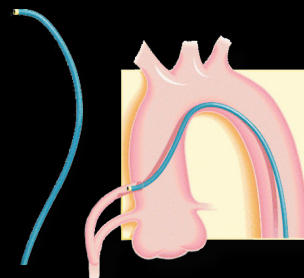


RCB

Right Coronary Artery via Femoral.

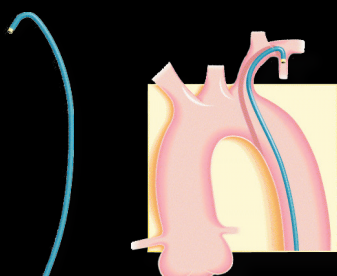

IM

Internal mammary femoral access.

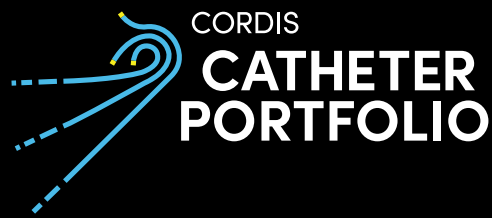

VISTA BRITE TIP™ Guiding Catheter





Back-up support and shape retention to help you through your challenging cases.

Available in a wide variety of shapes, including transradial-specific, and from 5F to 8F

- 1.** Back-up support and torque control
- 2.** Shape retention
- 3.** Easy delivery of device
- 4.** Enhanced visibility

Left Coronary Judkins Technique

JUDKINS LEFT




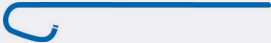
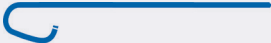
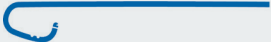
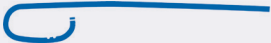
SHAPE					
Name	Picture	Observation	5F	6F	7F8F
JL 3				670-000-00	778-000-00
JL 3.5					778-002-00588-851
JL 3.5 LBT		Long VISTA BRITE TIP™	556-002-0L		
JL 3.5 ST		short tip		670-012-00	
JL 3.5 SH		2 side holes		670-003-00	778-003-00
JL 4				670-004-00	778-004-00588-823
JL 4		90 cm		670-004-90	
JL 4 LBT		Long VISTA BRITE TIP™	556-004-0L	670-004-0L	
JL 4 ST		short tip		670-014-00	778-014-00
JL 4 SH		2 side holes		670-005-00	778-005-00588-834
JL 4		125 cm		SM7435	
JL 4.5				670-006-00	778-006-00
JL 4.5 SH		2 side holes			778-007-00
JL 5				670-008-00	778-008-00
JL 5 LBT		Long VISTA BRITE TIP™	556-008-0L		
JL 5 SH		2 side holes			778-009-00
JL 6				670-010-00	778-010-00

JUDKINS CURVED LEFT

SHAPE				
Name	Picture	Observation	6F	
JCL 3.5			670-026-00	
JCL 4			670-028-00	

Left Coronary Extra Back-Up Shapes


EXTRA BACK-UP

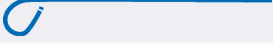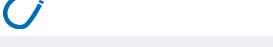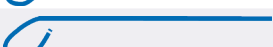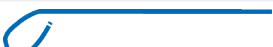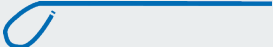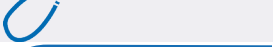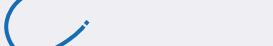Name	SHAPE		Observation	5F	6F	7F	8F
	Picture						
XB 2.5					SM7465*		
XB 3						778-052-00	588-829
XB 3 LBT		Long VISTA BRITE TIP™	SM7328				
XB 3 SH		2 side holes			670-053-00	778-053-00	
XB 3.5					670-054-00		588-882
XB 3.5		90 cm			670-054-90		
XB 3.5		115 cm			SM7747		
XB 3.5 LBT		Long VISTA BRITE TIP™	556-054-0L	670-054-0L			
XB 3.5 SH		2 side holes			778-055-00	588-885	
XB 4					778-056-00	588-894	
XB 4 LBT		Long VISTA BRITE TIP™	556-056-0L				
XB 4 SH		2 side holes			670-057-00	778-057-00	
XB 4.5					670-058-00	778-058-00	
XB 4.5 SH		2 side holes			670-059-00	778-059-00	
XBC 3					670-070-00		
XBC 3.5					670-072-00		
XBC 3.5 SH		2 side holes			670-073-00		
XBC 4					670-074-00		
XBLAD 3					670-066-00		
XBLAD 3 SH		2 side holes			670-067-00		

* Please note that the modified standard catheters will be produced on special request and are subject to longer delivery times and list prices.


Left Coronary Extra Back-Up Shapes

EXTRA BACK-UP

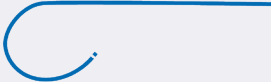Name	SHAPE		Observation				
	Picture			5F	6F	7F	8F
XBLAD 3.5 LBT			Long VISTA BRITE TIP™	556-060-0L			
XBLAD 3.5 SH			2 side holes		670-061-00	778-061-00	
XBLAD 4					670-062-00	778-062-00	
XBLAD 4 SH			2 side holes		670-063-00		
XBLAD 4.5					670-064-00		

Left Coronary Amplatz Technique

AMPLATZ LEFT

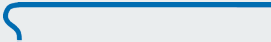Name	SHAPE		Observation				
	Picture			5F	6F	7F	8F
AL .75				SM7722		778-034-00	588-890
AL .75 SH			2 side holes		670-035-00		
AL 1						778-036-00	588-843
AL 1 LBT			Long VISTA BRITE TIP™	SM7331*			
AL 1 ST			short tip		670-044-00	778-044-00	
AL 1 SH			2 side holes		670-037-00	778-037-00	588-847
AL 1.5					670-038-00	778-038-00	
AL 2					670-040-00	778-040-00	588-844
AL2 LBT				556-040-0L	670-040-0L		

Left Coronary Amplatz Technique

AMPLATZ LEFT

SHAPE						
Name	Picture	Observation	5F	6F	7F	8F
AL 2 LBT		Long VISTA BRITE TIP™	556-040-0L	670-040-0L		
AL 2 SH		2 side holes		670-041-00	778-041-00	
AL 3				670-042-00	778-042-00	
AL 3 SH		2 side holes			778-043-00	

Right Coronary Judkins Technique

JUDKINS RIGHT

SHAPE						
Name	Picture	Observation	5F	6F	7F	8F
JR 3.5					778-080-00	588-855
JR 3.5 LBT		Long VISTA BRITE TIP™	556-080-0L			
JR 3.5 SH		2 side holes		670-081-00	778-081-00	
JR 4				670-082-00	778-082-00	588-830
JR 4		90 cm		670-082-90		
JR 4 LBT		Long VISTA BRITE TIP™	556-082-0L	670-082-0L		
JR 4 ST		short tip		670-090-00		
JR 4 SH		2 side holes			778-083-00	588-831
JR 4		125 cm		SM7436*		
JR 4		55 cm		670-082-55		
JR5				670-084-00	778-084-00	

* Please note that the modified standard catheters will be produced on special request and are subject to longer delivery times and list prices.

Right Coronary Extra Back-Up Shapes

RIGHT CORONARY EXTRA BACK-UP SHAPES

SHAPE				
Name	Picture	Observation	5F	6F
XBRCA SH		2 side holes		
				670-127-00

WILLIAMS TECHNIQUE

SHAPE				
Name	Picture	Observation	5F	6F
3DRC				670-130-00
3DRC LBT		Long VISTA BRITE TIP™	SM7599*	
3DRC SH		2 side holes		670-131-00

* Please note that the modified standard catheters will be produced on special request and are subject to longer delivery times and list prices.

Right Coronary Amplatz Technique

AMPLATZ RIGHT

SHAPE					
Name	Picture	Observation	5F	6F	7F
AR 1					778-110-00
AR 1 LBT		Long VISTA BRITE TIP™	SM7333*		
AR 1 SH		2 side holes		670-111-00	778-111-00
AR 2					778-112-00
AR 2 LBT		Long VISTA BRITE TIP™	556-112-0L	670-112-0L	
AR 2 SH		2 side holes		670-113-00	778-113-00


Transradial / Brachial Technique

TRANSRADIAL TECHNIQUE

SHAPE			
Name	Picture	Observation	
			5F6F
Barbeau		556-174-0L	670-174-00
JFL			670-162-00
JFR			670-168-00

Multipurpose

MULTIPURPOSE

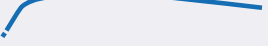SHAPE					
Name	Picture	Observation	5F	6F	7F8F
MPA 1 LBT		Long VISTA BRITE TIP™ 100 cm	556-270-0L	670-270-0L	
MPA 1					778-270-00588-842
MPA 1 SH		2 side holes		670-271-00	778-271-00
MPA 1		Multipurpose A 1 90 cm		670-270-90	778-270-90
MPA 1		125 cm		SM7477*	
MPA 2		125 cm		SM7394*	

HOCKEY STICK

SHAPE					
Name	Picture	Observation	5F	6F	7F8F
H-STICK				670-278-00	778-278-00588-841
H-STICK LBT		Long VISTA BRITE TIP™	556-278-0L		
H-STICK SH		2 side holes		670-279-00	778-279-00

Bypass

LEFT CORONARY BYPASS

SHAPE				
Name	Picture	Observation	5F	6F7F
LCB				670-180-00778-180-00
LCB SH		2 side holes		670-181-00

* Please note that the modified standard catheters will be produced on special request and are subject to longer delivery times and list prices.

Bypass

RIGHT CORONARY BYPASS

SHAPE					
Name	Picture	Observation	5F	6F	7F
RCB				670-182-00	778-182-00
RCB SH		2 side holes		670-183-00	778-183-00

INTERNAL MAMMARY

SHAPE						
Name	Picture	Observation	5F	6F	7F	8F
IM				670-190-00	778-190-00	
IM		90 cm		670-190-90		
IM SH		2 side holes		670-191-00	778-191-00	
IM LBT		Long VISTA BRITE TIP™	556-190-0L			
Tampa Bay		Tampa Bay				SM3371

Long Catheters

Long Catheters

Name	Observation	5F	6F	7F	8F
JL 4	125 cm		SM7435*		
JR 4	125 cm		SM7436*		
MPA 1	125 cm		SM7477*		
MPA 2	125 cm		SM7394*		

* Please note that the modified standard catheters will be produced on special request and are subject to longer delivery times and list prices.

Long VISTA BRITE TIP™ Guiding Catheters

Long VISTA BRITE TIP™ Guiding Catheters

- Available in 6F .070" & 5F .056"
- Active back up support through deep seating
- Atraumatic ostial coaxial alignment by the excellent tip conformability
- Good tip visibility
- Atraumatic TIP

LEFT CORONARY JUDKINS TECHNIQUE

JUDKINS LEFT

Name	Observation	5F	6F
JL 3.5		556-002-0L	
JL 4		556-004-0L	670-004-0L
JL 5		556-008-0L	

LEFT CORONARY EXTRA BACK-UP SHAPES

EXTRA BACK-UP

Name	Observation	5F	6F
XB 3.5		556-054-0L	670-054-0L
XB 4		556-056-0L	
XBLAD 3.5		556-060-0L	

LEFT CORONARY AMPLATZ TECHNIQUE

AMPLATZ LEFT

Name	Observation	5F	6F
AL 1		SM7331	
AL 2		556-040-0L	670-040-0L

RIGHT CORONARY JUDKINS TECHNIQUE

JUDKINS RIGHT

Name	Observation	5F	6F
JR 3.5		556-080-0L	
JR 4		556-082-0L	670-082-0L

WILLIAMS TECHNIQUE

Name	Observation	5F
3DRC		SM7599

* Please note that the modified standard catheters will be produced on special request and are subject to longer delivery times and list prices.

Long VISTA BRITE TIP™ Guiding Catheters

RIGHT CORONARY AMPLATZ TECHNIQUE

AMPLATZ RIGHT

Name	Observation	5F	6F
AR 1		SM7333*	
AR2		556-112-0L	670-112-0L

MULTIPURPOSE

MULTIPURPOSE A 1

Name	Observation	5F	6F
MPA 1		556-270-0L	670-270-0L

HOCKEY STICK

Name	Observation	5F	6F
H-STICK LBT		556-278-0L	670-278-0L

BYPASS

INTERNAL MAMMARY

Name	Observation	5F	6F
IM		556-190-0L	

* Please note that the modified standard catheters will be produced on special request and are subject to longer delivery times and list prices.

ECO-PAC FIVE PACS

LEFT CORONARY JUDKINS TECHNIQUE

Name	Observation	5F	6F	7F	8F
JL 3.5	Judkins Left 3.5				
JL 4	Judkins Left 4				

670-002-00M

670-004-0E

MULTIPURPOSE A 1

Name	Observation	5F	6F	7F	8F
MPA 1	Multipurpose A 1 90 cm				
MPA 1	Multipurpose A 1				

588-842PM

670-270-00M

AMPLATZ LEFT

Name	Observation	5F	6F	7F	8F
AL .75	Amplatz Left 0.75				
AL 1	Amplatz Left 1				

670-034-00M

670-036-00M

AMPLATZ RIGHT

Name	Observation	5F	6F	7F	8F
AR 1	Amplatz Right 1				
AR 2	Amplatz Right 2				

670-110-00M

670-112-00M

JUDKINS RIGHT

Name	Observation	5F	6F	7F	8F
JR 3.5	Judkins Left 3.5				
JR 4 SH	Judkins Right 4, 2 side holes				
JR 4	Judkins Right 4				

670-080-00M

670-083-00M

670-082-0E

EXTRA BACK-UP LEFT

Name	Observation	5F	6F	7F	8F
XB 3	Extra Back-Up 3				
XB 3.5	Extra Back-Up 3.5				
XB 3.5 SH	Extra Back-Up 3.5, 2 side holes				
XB 4	Extra Back-Up 4				

670-052-00M

670-054-0E 778-054-00M

670-055-00M

670-056-00M

EXTRA BACK-UP LAD

Name	Observation	5F	6F	7F	8F
XB LAD 3.5	Extra Back-Up LAD 3.5				

670-060-00M

EXTRA BACK-UP RCA

Name	Observation	5F	6F	7F	8F
XB RCA	Extra Back-up Right CA				

670-126-00M

For Healthcare Professionals Only.
Important information: Prior to use, refer to the instructions for use supplied with this device for indications, contraindications, suggested procedure, warnings and precautions. As part of its continuous product development policy, Cordis reserves the right to change product specifications without prior notification.
Please contact your Cordis representative for additional product availability information.
CORDIS, Cordis LOGO and VISTA BRITE TIP are trademarks of Cordis and may be registered in the US and/or in other countries.
© 2025 Cordis. All Rights Reserved. 100552147-5

